


How to Use the Topi-CLICK[®] PERL[™] Vaginal Dosing Kit



A. Opening/Attaching the Snap-on Applicator to the PERL

1. Flip PERL shell cap open using integrated tab (*Figure a.*)
2. Snap the Snap-on Applicator to the extended receptor port on the PERL shell. (*Figure b.*) **Note:** The Snap-on Applicator Plunger should be fully inside the applicator barrel when dosing as shown. (*Figure b and c.*)

Snap-on Applicator[™]

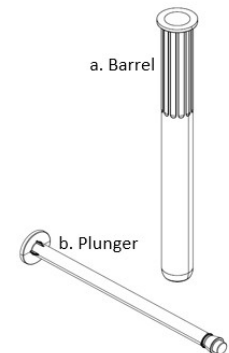

B. Loading PERL / Dispensing Medication

1. To dispense medication, simply turn/CLICK the base of the PERL to load the metered dose – as with all Topi-CLICKs, wait 15 to 30 seconds for full dose to load into the Snap-on Applicator. (*Figure c.*)

*** It is important to keep the PERL in vertical position when loading the Snap-on Applicator[™] as shown. (*Figure c.*)**

Dosing Notes:

Each turn/CLICK dispenses 0.25 mL of medication accurately and does not need fill lines on the Snap-on Applicator. (see table below)

Dosing Guide	
1 CLICK	= 0.25 mL
2 CLICKs	= 0.50 mL
3 CLICKs	= 0.75 mL
4 CLICKs	= 1.00 mL


No Reference Lines on Applicator:

The Topi-CLICK is the most accurate** topical dispenser available. Unlike the traditional tube and applicator, the Snap-on Applicator was designed to be loaded by Topi-CLICK technology and therefore does not need reference lines to ensure accuracy.


** Go to www.doselogix.com/accuracy_study to request a free copy of the independent accuracy study.



"Flip Open" to Open

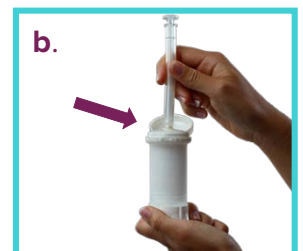

"Snap-On" to Load

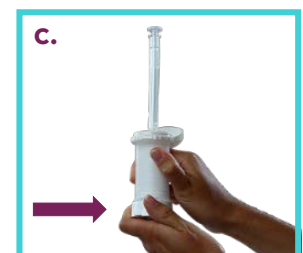

"Turn / CLICK" to Dispense



"Bend-Off" to Remove

C. Removing the Snap-on Applicator

1. Simply lean the Snap-on Applicator over as shown in (*Figure d.*) until the applicator releases from the receptor extended port on the shell.
2. **Reminder:** Depending on the patients removal technique, there may be small dollop of cream remaining on the Snap-on Port which is part of the patients dose.



- ◆ USP Class VI compliant
- ◆ BPA/Latex Free
- ◆ Hand wash applicator with warm water only; not dishwasher safe.

Rev. 11042015

Topi-CLICK® Filling Instructions



STEP 1:
Fill the Topi-Click® container with the desired amount of topical preparation using a spatula, EMP jar, or filling system.



STEP 2:
If needed while filling, stop periodically to tap the Topi-Click® on the palm of your hand or counter to help settle the topical.



STEP 3:
Put the applicator pad (cap with a hole) on the top of the Topi-Click® container by aligning the inside key tabs with the slots on the top of the container tube and snap on.



STEP 4:
Prime the Topi-Click® by turning the bottom until topical is dispensed and wipe off topical.

Be sure to leave the base in the registered click position by aligning the base column with the tube node: See Step 3 of the Operating Instructions.



STEP 5:
Snap on the protective outer cap. (if the cap doesn't fit tight, it's an indication that the applicator pad in #2 is not "snapped" down all the way).



STEP 6:
Apply center of label to the side of the Topi-Click® container which is opposite the "REFILL" and graduation marks.