



MEDISCA[®]
YOUR TRUSTED PARTNER IN COMPOUNDING

PRODUCT NO.: 7816







MEDISCA ADAPTER CAP (E, 28mm)

28-400, LDPE, Red (Short Neck)

DESCRIPTION: Match the bottle closure-code with the MEDISCA Adapter Cap of your choice for easy connection to bottle tops for simplified oral syringe filling. Please contact a MEDISCA Sales representative for your very own MEDISCA Adaptor-Chek, for quick identification of bottle threads.

FEATURES:

-  Made out of low-density polyethylene (LDPE) with a tethered, re-closable cap
-  Non-DEHP
-  Used on most glass or plastic bottles to draw liquid
-  Latex-free


ADDITIONAL INFORMATION:

Closure Code:

Letter	Closure Code	mm Code	GPI/SPI Neck Finish
A	18-400	18	400
B	20-400	20	400
C	22-415	22	415
D	24-410	24	410
M	24-415	24	415
E	28-400	28	400
K	28-410	28	410
F	28-415	28	415

Closure Code Explanation:

- 1) The closure code consists of the cap diameter (mm) and neck finish.
- 2) The neck finishes that are the most common (and that we will work with) are 400, 410, and 415. If you match a bottle neck below with the picture, you will be able to tell the finish.

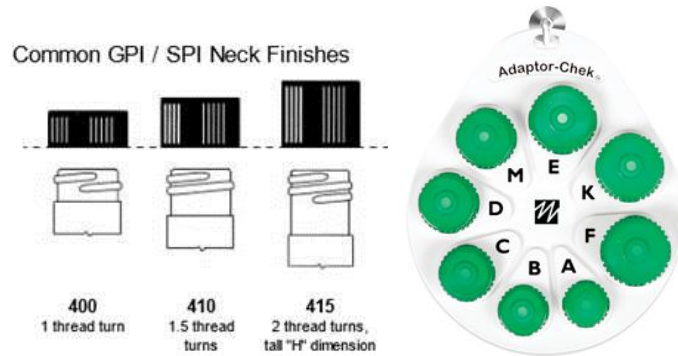

- 3) The closure codes must match, e.g., a 24-410 bottle will only fit a 24-410 cap (adapter-cap D).
- 4) You will see that some Adapter-caps share the same diameter and thus can be easily confused even when using the Adaptor-Chek (seen above). Problems will arise if a D adapter-cap is being used instead of an M, or the like.